

Small Fold Down Trunk Lifter – 4 Way

Small Fold Down Trunk Lifter is quick, effortless loading of your scooter or wheelchair



How To Use Small 4 Way Trunk Lifter Hoist:

- 1 With the scooter/wheelchair attach the bracket to the scooter
- 2 Simply press the **Up** button & the scooter will be lifted



- 3 Once the scooter has reached the correct height, press the **In** button & the scooter can be manoeuvred into the vehicle
- 4 Press the **Down** button to lower the scooter into the position



Please check with Frank's Engineering Staff, as assessment of vehicle & scooter/wheelchair may be required.

Frank's Engineering Pty Ltd

135 Bakers Road North Coburg Vic 3058

Ph: 03 9354 0400 | Fax: 03 9354 0344

Email for Quotes: sales@franksengineering.com.au

www.franksengineering.com.au